TTWB24H

TTLAPIV72H

TTLAUC72H

TTLA96H

TTLA7D

EN

Description Label alerts that indicate a set time period has elapsed.

Start Pull out activation tag. Green light flashes for 30 seconds, then once every 10 minutes.

Place Remove release liner from rear of label and affix.

Alert Red light starts flashing 6 hours before time period printed on label/wristband.

Disposal Please recycle by disposing into battery recycling container. RoHS compliant in accordance with EU Directive 2011/65/EU.

Shelf Life At least 5 years from date of manufacture.

MRI/XRay - TTWB24H Tear at perforation marked MRI/X-Ray to remove. After MRI/X-Ray reattach using remaining section of adhesive.

Performance Minimum alert duration 24 hours. Median error (ME)=0.5%. Maximum observed error (MOE)=1.5%. Example: TTLAPIV72HR ME=20mins MOE=60mins.

Warnings Avoid X-Rays or Magnetic Resonance Imaging (MRI). Contains a lithium manganese battery. Seek medical attention if swallowed. Keep out of reach of children. Do not burn, heat above 100°C (212°F), puncture or crush - explosion risk. Timertag products are workflow aides and do not prevent disease - no health benefit claims are made.

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