

# SAM Safety Shield Guedel

SAM

SCOTT AIRWAY MANAGEMENT

The **SAM Safety Shield** is designed to be used together with the **SAM Guedel** during bag mask ventilation with bearded, misshapen face and denture-less patients to assist in providing a better seal. The device is also used with any patient where aerosolisation of patient secretions during bag mask ventilation is a concern.

The device is easy to use. Simply invert the **SAM Safety Shield** wings into a gloved hand and place the guedel as per standard practice. Once the guedel is in situ, release the wings so that they lay onto the patient's face.

Once the bag mask ventilation process is completed, simply grab hold of the **SAM Safety Shield** wings and fold around the guedel into your gloved hand and discard into an appropriate receptacle.

Scan the QR Code to watch a video on how to use the **SAM Safety Shield** products.

## Product Specifications

Silicone - NON Latex

Body Height 2cm


SINGLE USE ONLY

Silicone Variant thickness - 1.5-2.5mm

Wing Dimensions 17cmx10cm

SAM SAFETY SEAL ARTG 331 999

SAM GUEDEL AIRWAY ARTG 353 530

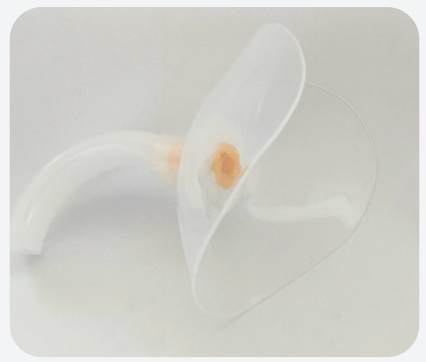

### Product

### Product Order Code

### Quantity

SAM Safety Shield Guedel 9cm (Yellow)

SSSGN09

50

SAM Safety Shield Guedel 10cm (Red)

SSSGN10

50



**LATERAL**  
**MEDICAL**

**Lateral Medical**

DISTRIBUTOR QLD/VIC

1300 144 734

sales@lateralmedical.com